


# GUIDELINE TO DOWNLOAD AND INSTALL THE LIBRARY OF CAD FILES OF 3.0 DYNAMIC TITANIUM BASES FOR 3SHAPE

3shape 




1. Request the files for 3Shape to the following e-mail: [das@dynamicabutment.com](mailto:das@dynamicabutment.com), including your contact details. Enter in the Dropbox link provided by Dynamic Abutment Solutions and copy “DAS\_3SHAPE\_V5.rar” folder (V5 can change depending on the version)

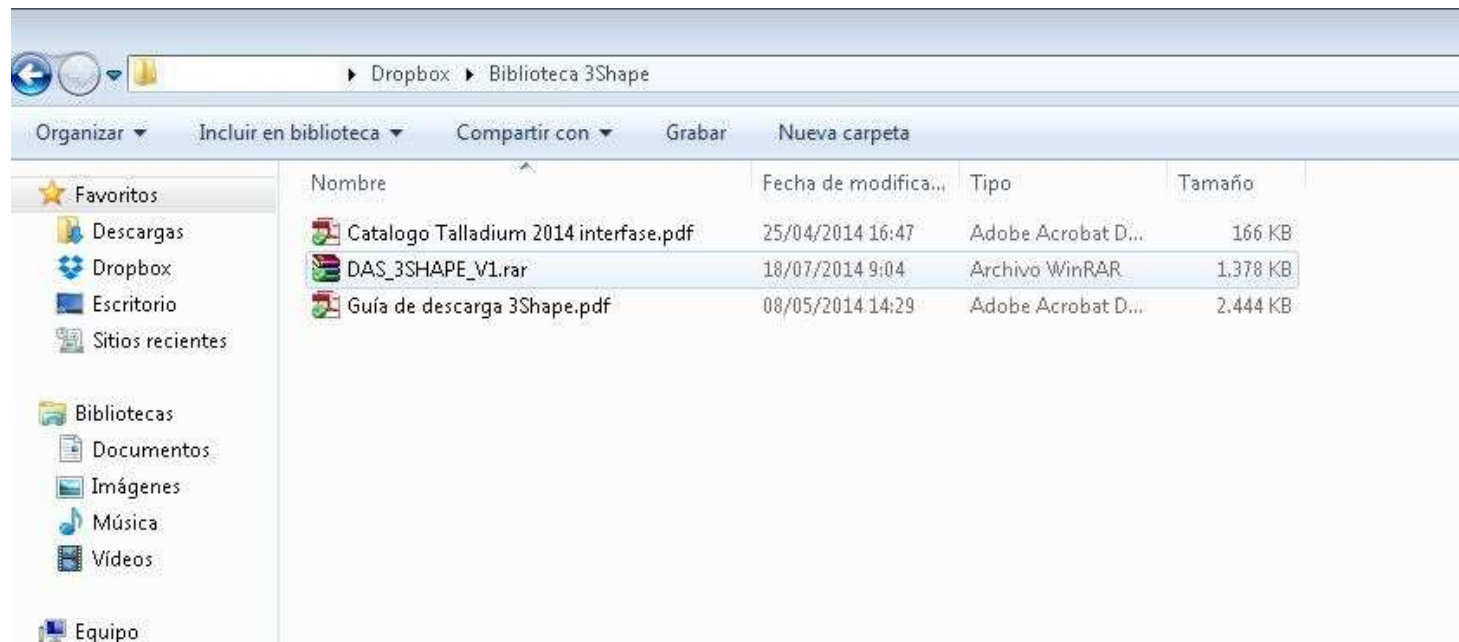


2. Paste .rar file and unzip wherever you want, for example at desktop, and you will get the folder “DAS\_3SHAPE\_V5”.



3. After unzip the file, check that the unzip folder contains the file “DAS\_3SHAPE\_V5.dme”, which is an only file that contains the components for 3SHAPE.

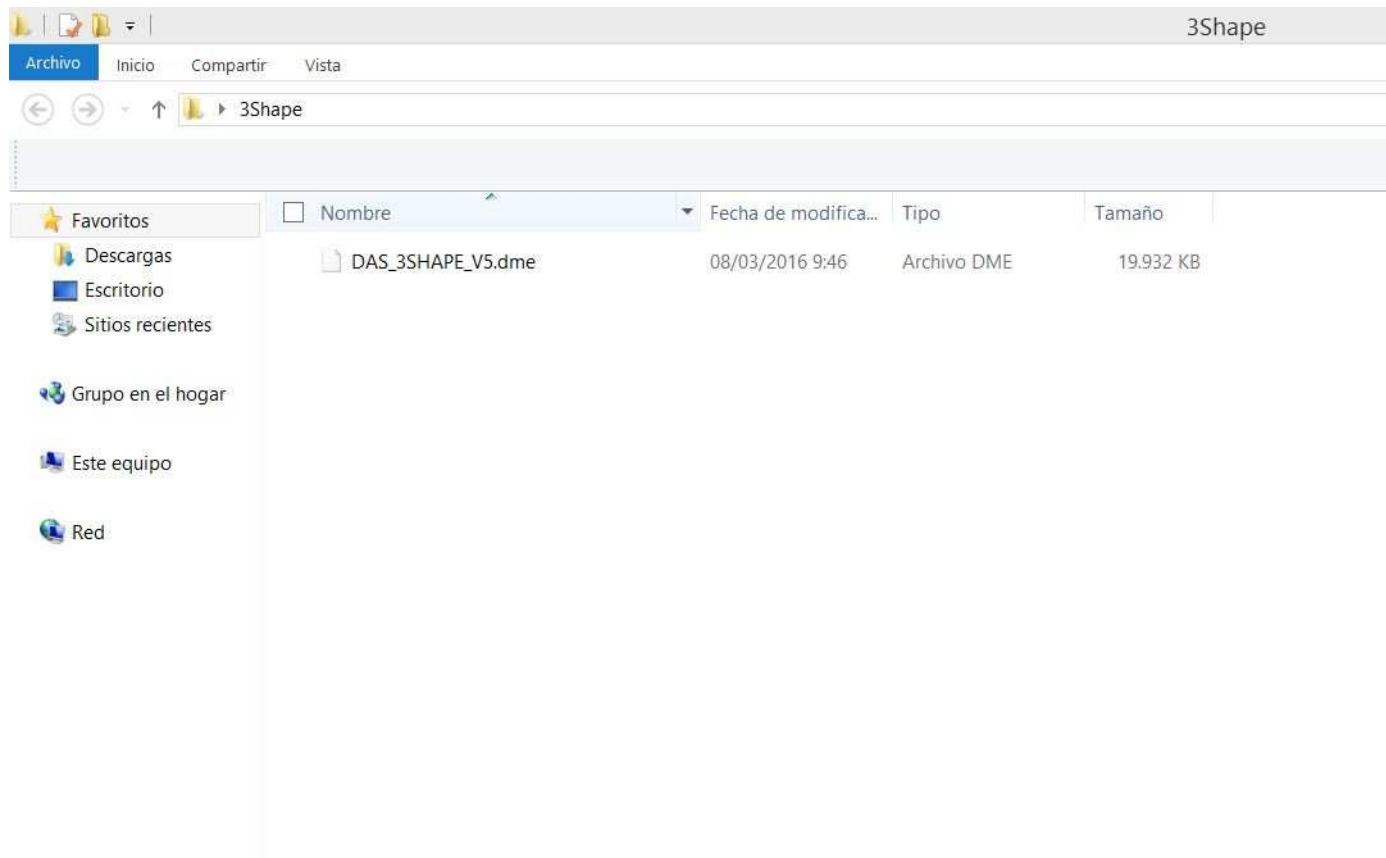

## 4. Start 3SHAPE Dental System program and go to Tools Import/Export.

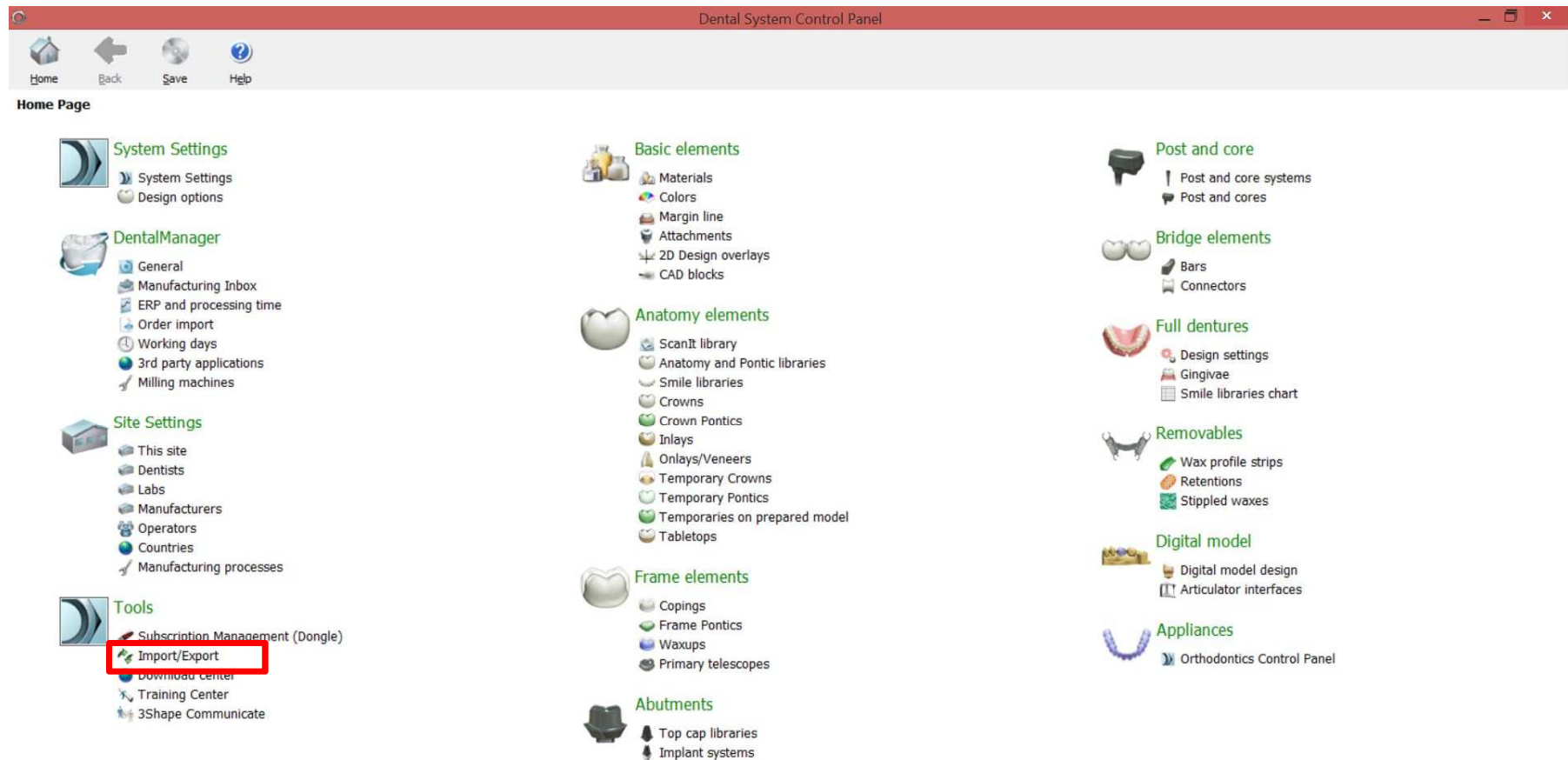

# 5. Select Import Materials.

Dental System Control Panel - Export/Import Materials

Home Back Save Help

Tools > Import/Export

Tasks

- Export materials
- Import materials

Import materials

To import new materials, click on "Import materials" and browse the desired file (extension ".dme"). Then select the materials from the list that appeared on a screen and click "Import". Typically it is recommended to select all. You can also download the materials directly from FTP - go to Site Settings -> Sites page and click "Download Materials".

Import materials

History Of Import Materials

Export materials

Clone system

6. When windows explorer is opened, look for the file extension .dme “DAS\_3SHAPE\_V5.dme”, select the file and click on open.


The screenshot shows the 'Dental System Control Panel - Export/Import Materials' web interface. On the left, there are navigation buttons (Home, Back, Save, Help) and a 'Tools > Import/Export' section with 'Tasks' for 'Export materials' and 'Import materials'. The main content area has an 'Import materials' button and instructions: 'To import new materials, click on "Import materials" and browse the desired file (extension ".dme"). Then select the materials from the list that appeared on a screen and click "Import". Typically it is recommended to select all. You can also download the materials directly from FTP - go to Site Settings -> Sites page and click "Download Materials".' Below this is a 'History Of Import Materials' section and buttons for 'Export materials' and 'Clone system'.

Overlaid on the interface is a Windows Explorer window titled 'Abrir' showing the '3Shape' folder. The file list contains one entry:

Nombre	Fecha de modifica...	Tipo	Tamaño
DAS_3SHAPE_V5.dme	08/03/2016 9:46	Archivo DME	19.932 KB

The Explorer window also shows a search bar 'Buscar en 3Shape', a 'Nombre:' field with a dropdown menu set to 'Dental System Materials Export', and 'Abrir' and 'Cancelar' buttons.

# 7. Titanium bases and their components library is ready to use.

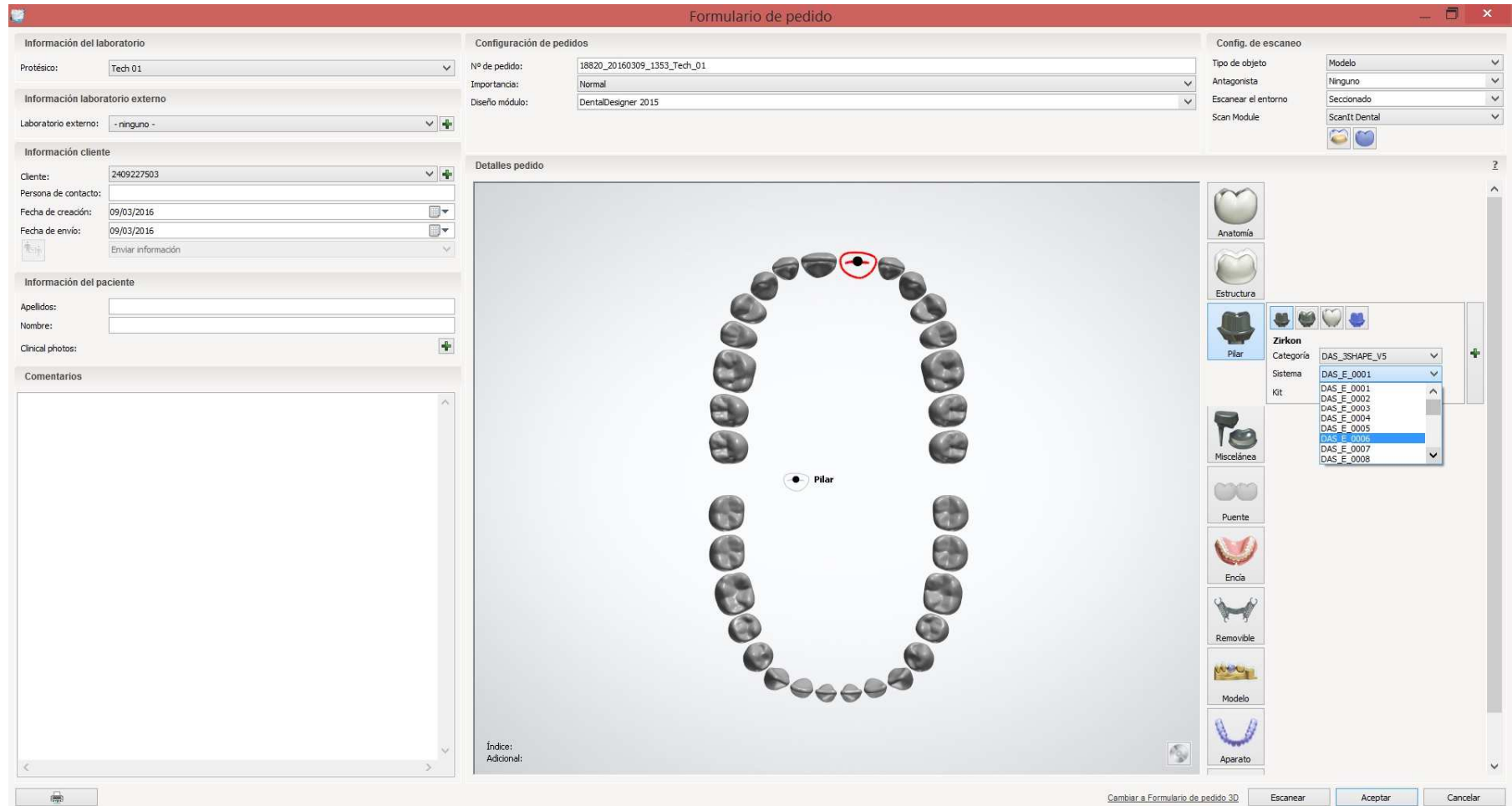



Note: If you already have a DAS library version, remove completely the file .dme and install the new library following the instructions.

