


GUIDELINE TO DOWNLOAD AND INSTALL THE LIBRARY OF CAD FILES OF 3.0 DYNAMIC TITANIUM BASES FOR EXOCAD

exocad




1. Request the files for Exocad to the following e-mail: *das@dynamicabutment.com*, including your contact details. Enter in the Dropbox link provided by Dynamic Abutment Solutions and copy “DAS_EXOCAD_VO4_signed.7z.rar” folder.

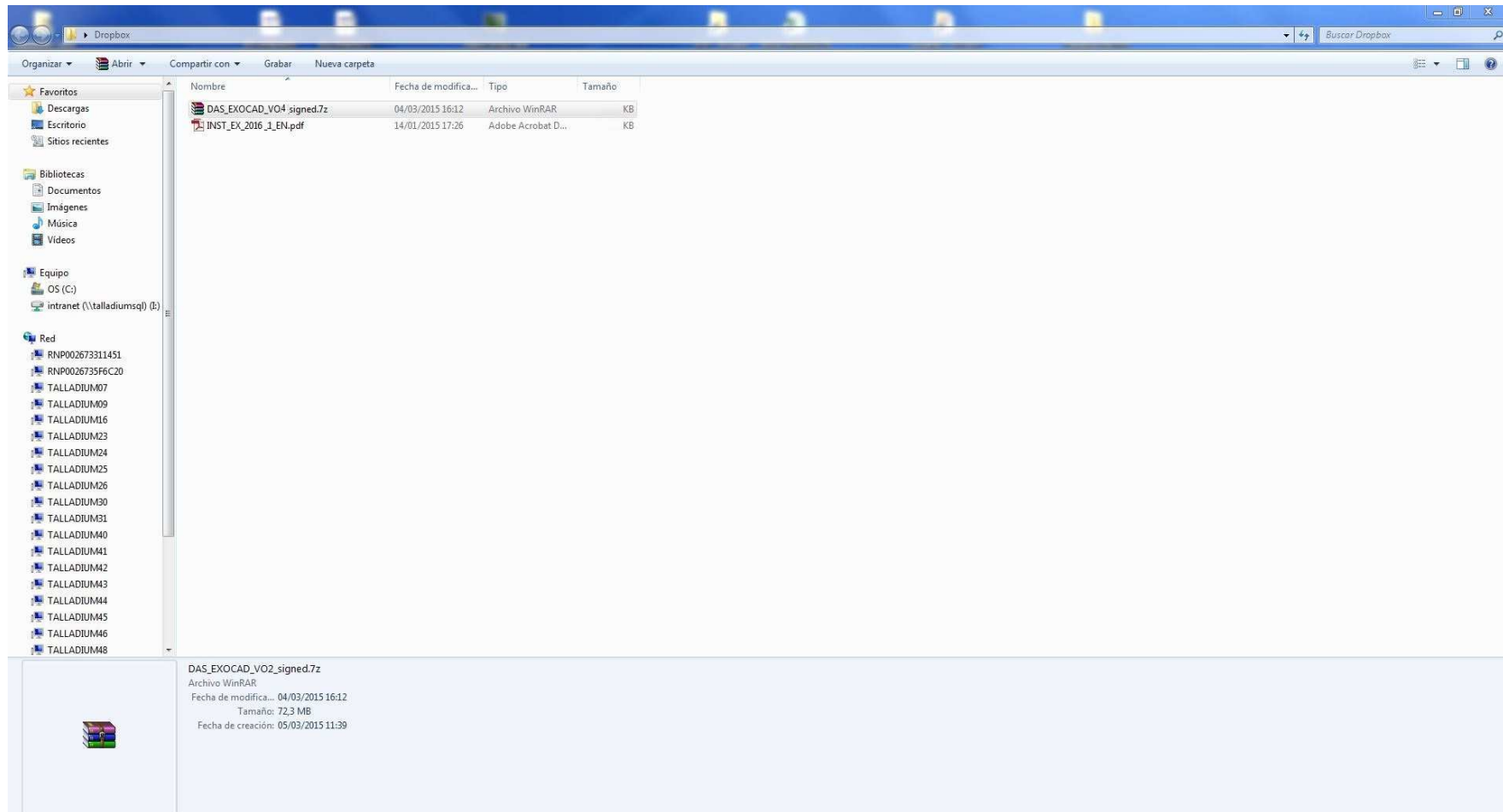

2. Paste .rar file and unzip wherever you want, for example at desktop, and you will get the folder “DAS_EXOCAD_VO4_signed.7z”.



3. After copy the folders that are in “DAS_EXOCAD_VO4”, and paste them in the following index: “Exocad-DentalCAD/DentalCADApp/library/implant”.

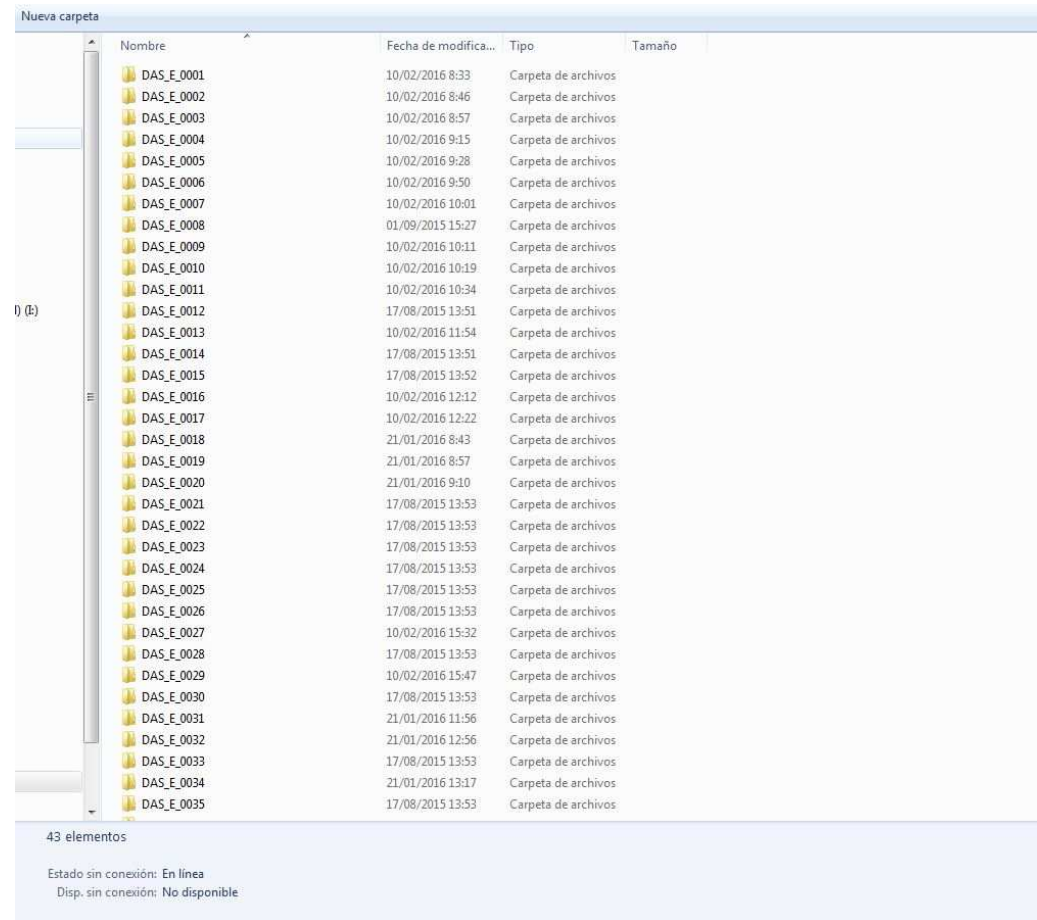

Note: If you already have a DAS library version, remove completely the files and install the new library following the instructions.



4. Start Exocad with a new project and verify that the libraries are installed.

