

Bidra Conversion Smart Polishing Cap

1. Screw the Bidra Conversion Smart Polishing Cap into the metal coping
2. Begin the conversion process by adding resin (**See Figure 1**)

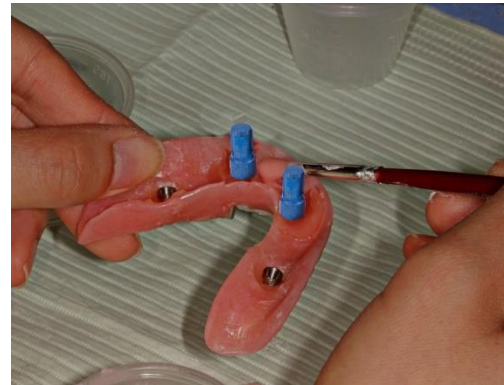


Figure 1



Figure 2

3. Allow acrylic resin to completely set (**See Figure 2**)
4. Watch the Bidra Conversion Smart Polishing Cap in action!

5. If applicable: after anterior acrylic has set, remove and screw Bidra Conversion Smart Polishing Caps onto posterior copings

6. Repeat Steps 2-4 (**See Figure 3**)



Figure 3