



## Fiberglass Pencil

[www.preat.com](http://www.preat.com)  
800-232-7732



### Technical Information

The holder for the fiberglass work like a mechanical pencil, and by turning the end of the pencil the user can extend or reduce the exposed fiberglass insert. A new fiberglass refill insert may be inserted into the pencil when needed.

The fiberglass insert is designed for **removal of investment** from castings and the **removal of oxides or other discolorizations** from cast or machined metal.

The user should use this instrument **in front of a vacuum and wear both protective glasses and breathing masks** to eliminate either breathing or absorbing the airborne fiberglass particles.

T0097.REV00