



www.preat.com
800-232-7732

Goats Hair Wheels

Technical Information



The goat hair brushes are made of goat hair bristles and an interior of micro fiber leather. The goat hair bristles are very soft for a final polish and elimination of scratches. The interior micro fiber leather provides form and position for the goat hair bristles to extend the life and ease of use of the brushes.

A small amount of polish ([PREAT 244 Blue](#) is recommended) is placed on the brush while it rotates at a very slow speed.

At low to moderate pressure run the lathe at 1000- 5000RPM for final polishing.

The user should wear protective eyewear and breathing protection and use in front of vacuum.

T0101.REV00