


Preci Post Instructions

1

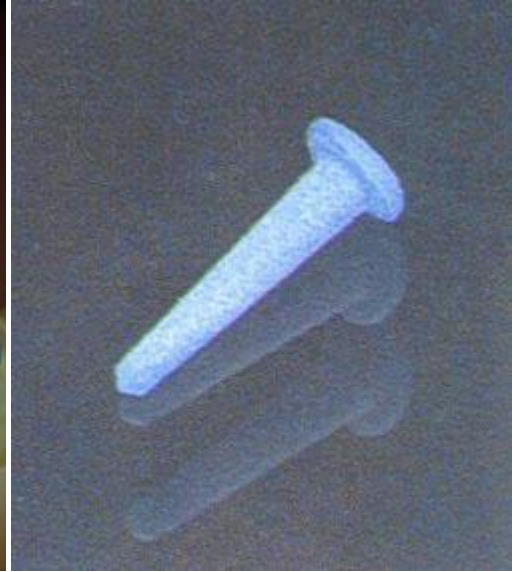

2



1. Prepare the root canal with either the Yellow or Blue Root Canal Reamer (**FIG 2**). Roughen the posts to provide retention (**FIG 2**).

3




4




2. Place the corresponding yellow or blue opaque impression post in the canal (**FIG 3**).
3. Take an accurate impression. The posts are provided with retentions for a proper fit in the impression material and will allow it to be draw off with the impression (**FIG 4-5**).




4. Pour the master model and prepare dies. The Impression post is self-separating and will not adhere to most die materials (**FIG 6**).

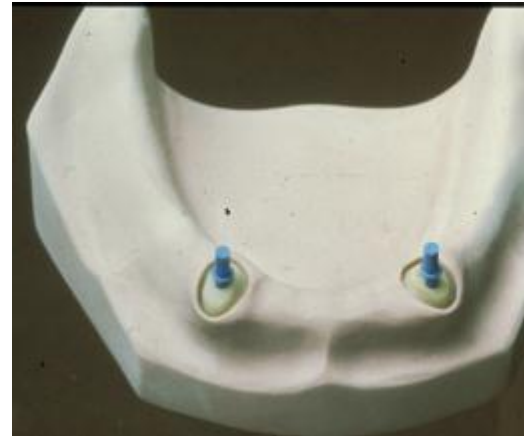

5. In the Laboratory, remove the impression post and replace it with the transparent laboratory burn-out post (**FIG 7**). Reduce it to its correct length and make sure that the top offers a little retention for wax.

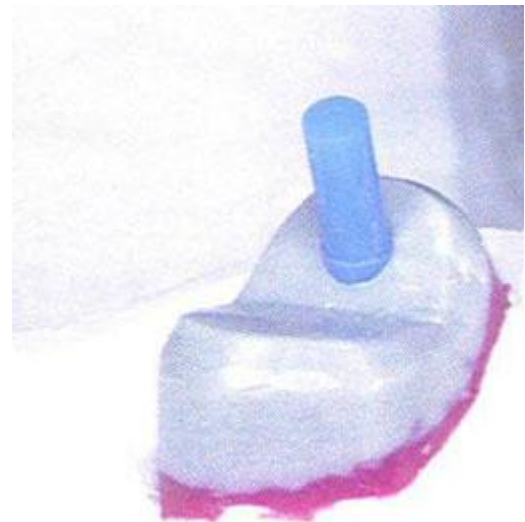

6. Make the wax pattern directly to the post (**FIG 8**).

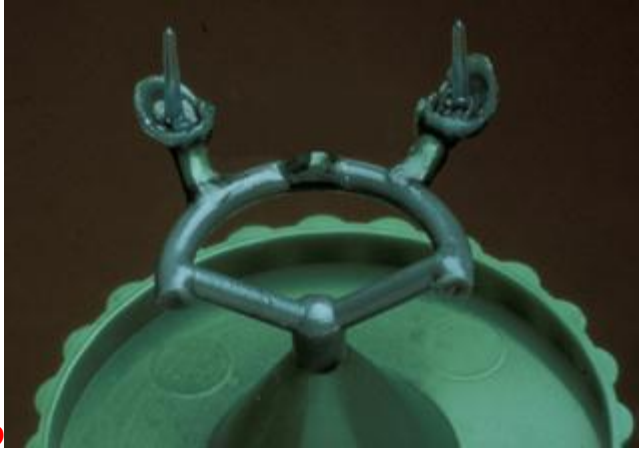

9



10

7. Sprue, invest (Preci-Roto System), burnout, and cast the pattern (**FIG 9**). The result is always an exact fitting post-coping without solder joint (**FIG 10**).

T0077.REV.00