




# TOOLS FOR THE SPHERO BLOCK ATTACHMENTS

[www.preat.com](http://www.preat.com)  
800-232-7732

To insert the sphere of the attachment onto the tool:

- Push the small piston down, on the opposite side, hexagonal pliers will emerge

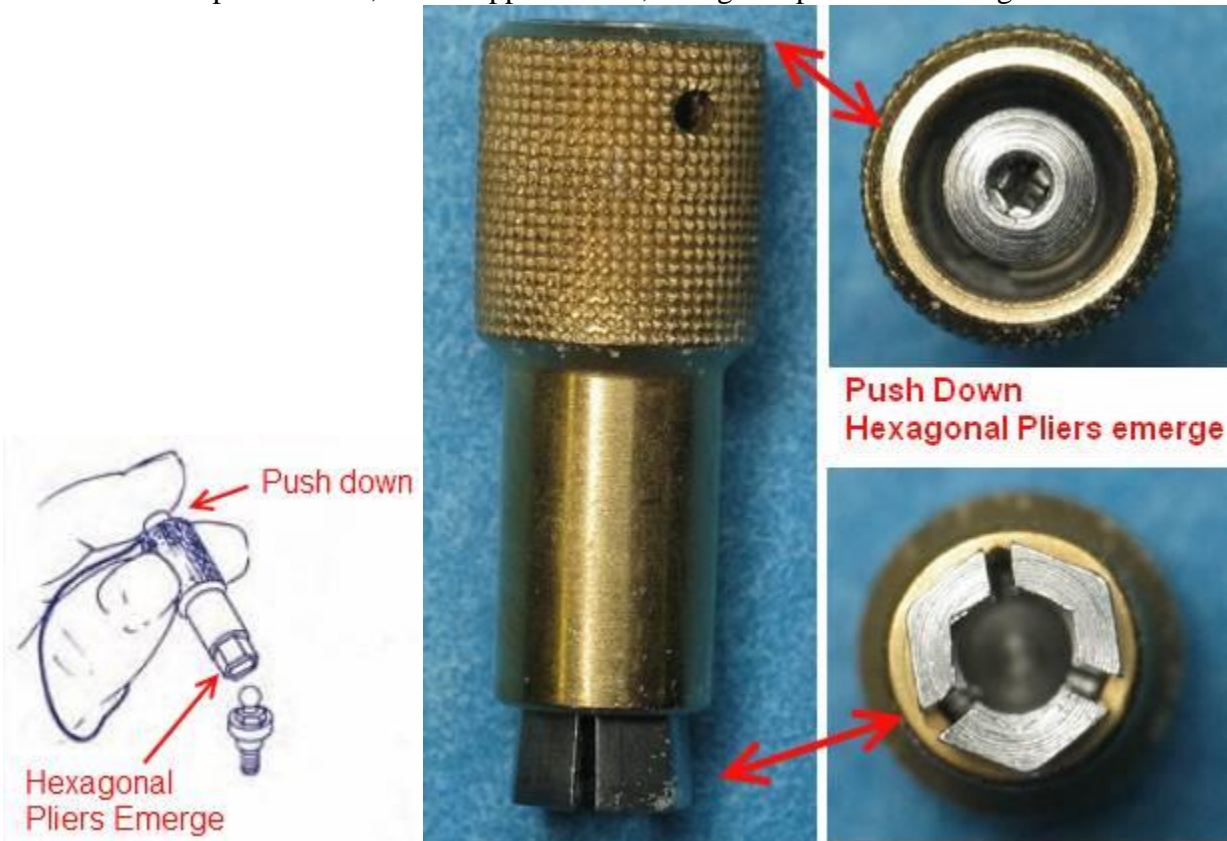

- Insert the sphere into the tool. The sphere will bypass the “loose” hexagonal pliers. Make sure that the hexagonal pliers are correctly oriented around the hex underneath the sphere.



- Pushing the external part of the tool down, the hexagonal pliers closes and holds the attachment



- The tool has a .050 hex on top. Use any .050 driver with your torque wrench to torque the abutments into place. 25ncm is recommended.

