




[www.preat.com](http://www.preat.com)  
800-232-7732

## Silicone Cutters and Polishers

### Technical Information

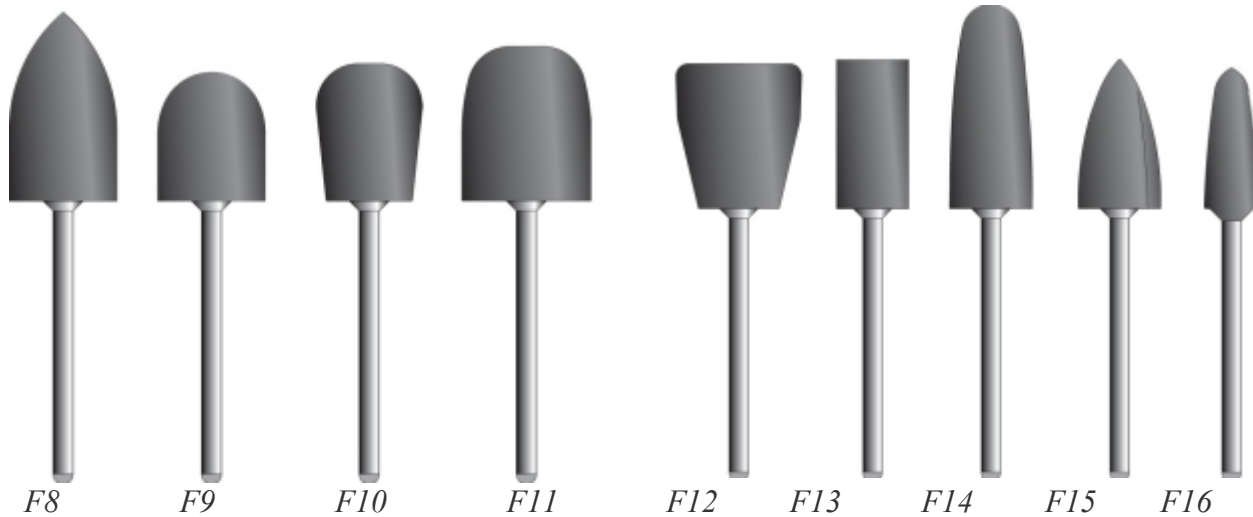

Silicone Cutter Polishers are indicated for adjusting and polishing acrylic denture base resins. They may be used in either a slow speed straight handpiece or a dental lathe.

The polishers should be used at speeds of 10,000 to 15,000 RPM with moderate pressure (between 0.3 and 2 N).

The polisher's consist of silicone rubber, pigments, and abrasives. The shanks are stainless steel.

The Silicone Cutter Polishers are not intended for use in the mouth. Do not exceed 15,000 RPM. Breathing and eye protection should be worn during use.